
User Manual

Version 2.0

ACE mini Incubator (For ACE test® Biological Indicator Only)

Catalog Number:

FUK-H8200

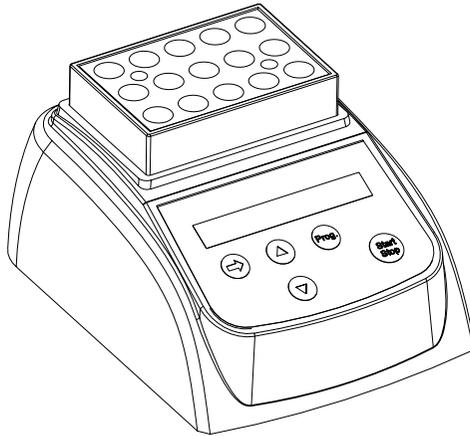Thank you for purchasing the ACE mini Incubator.

Please make sure to read this manual thoroughly before use.
After reading, please keep this manual in a safe place for future reference.

Revised Dec. 2015

COSMO BIO Co., LTD.
Inspiration for Life Science

Table of Contents

Safety Precautions	1
Package Contents / Component List.....	1
Names of Components.....	2
Display.....	3
Specifications.....	3
Operation Guide.....	4
Temperature Calibration.....	6
Troubleshooting.....	8
Maintenance	8

Safety Precautions

Warning

- Do not use with any voltage other than specified.
- Only use the included AC adapter. Do not use other adapters.
- Keep water and other liquids out of the unit.
- Do not operate near flammable liquids or gases.
- Keep away from open flames. Also, do not place near heaters, stoves, or in direct sunlight.
- Never attempt to modify or repair the unit yourself.
- Do not touch the plug or switches with wet hands.
- If the power cord is damaged, stop using it immediately.



Caution

- Do not damage, modify, forcibly bend, pull, twist, or bundle the power cord.
- Avoid dropping or tipping the unit, and do not subject it to strong impacts.
- Do not place the device in areas with excessive moisture, dust, or where gases may be present.
- If you plan not to use the machine for an extended period, unplug it from the outlet.


Main Unit and Accessories

Main Unit	Incubator ×1 pc
Accessories	Clear Insulation Lid ×1 piece Power Cord ×1 unit AC Adapter ×1 unit Tool Set ×1set User Manual ×1 booklet Warranty Card ×1 sheet Inspection Certificate ×1 sheet

Parts & Components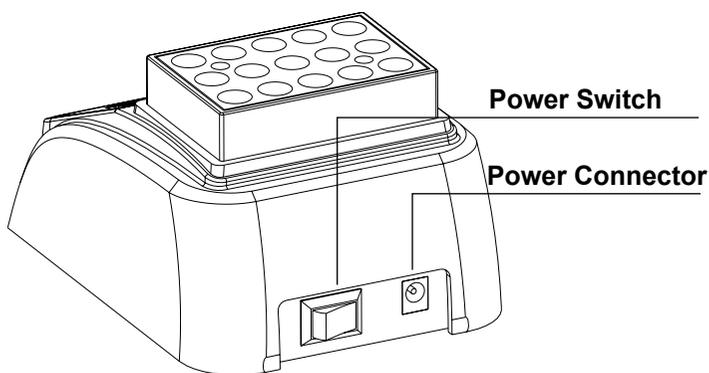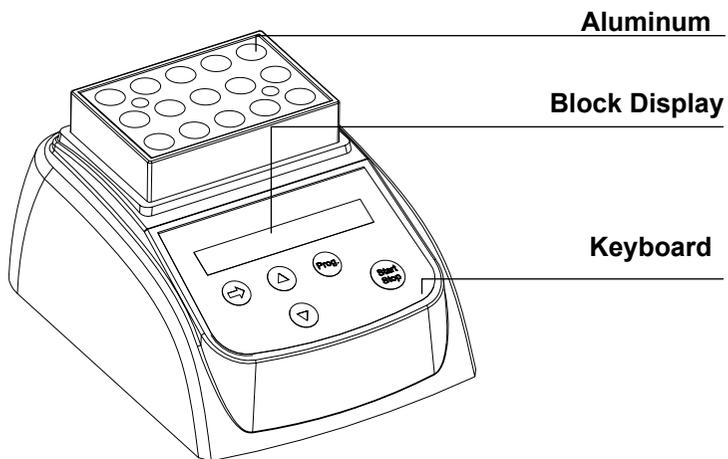

Display

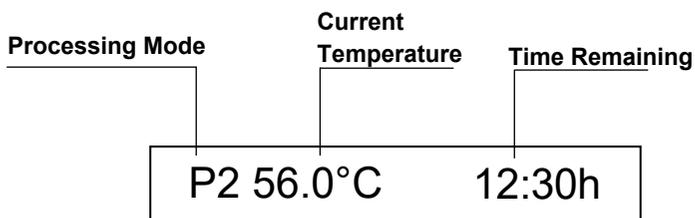Specifications

Range Operating Temperature	Room temperature +5°C to 80°C
Heating Time	Within 12 minutes (20°C→80°C)
Temperature Accuracy	±0.5°C
Timer Setting	0 min to 99 hr 59 min
Display unit	0.1°C
Thermoblock material	Aluminum
Operating environment	5°C to 35°C
Dimensions	W110mm × D150mm × H110mm (with lid)
Weight (including block)	0.5 kg
Power Consumption	Max: 35W, Avg: 5W
Power Supply	AC100~240V, 50/60Hz, 80VA

Operation Guide


This device can store up to 9 programs combining temperature and time. The preset temperature and time combinations are saved as P1, P2, ... P9.

Initial Setup

Program	Temp	Time	Applicable Sterilization Methods
P1	56.0°C	24:00h	Steam, hydrogen peroxide, LTSF, formalin (Biological indicator: Geobacillus stearothermophilus)
P2	56.0°C	48:00h	
P3	56.0°C	0:00h	
P4	35.0°C	24:00h	EO gas, dry heat, formalin, Chlorine dioxide (Biological indicator: Bacillus atrophaeus)
P5	35.0°C	48:00h	
P6	35.0°C	0:00h	
P7	37.0°C	24:00h	115°C Steam (Biological indicator: Bacillus subtilis)
P8	37.0°C	48:00h	
P9	37.0C	0:00h	

Note: "00:00h" indicates that no time has been entered yet.


1. Switch on the device. A beep will sound and the display on the right will show the current status.
2. Next, P1 will appear on the display. In the example on the right, "35.5°C" shows the current temperature of the aluminum block. The machine will automatically heat up. "NOTOK" will be displayed on the right side of the screen.
3. "Prog." button. Then select the desired program (P1, P2, ... P9) for your preferred temperature and time. The machine will gradually heat up or cool down. When the set temperature is reached, a beep will sound and "OK" will appear on the right.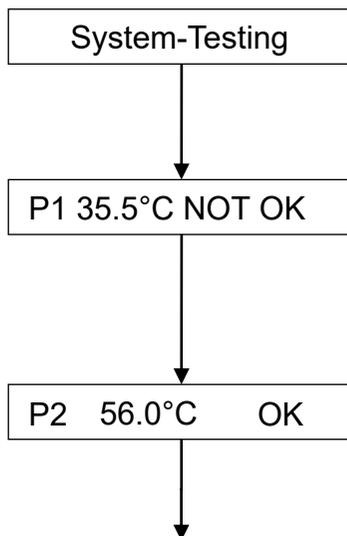

4. Insert the ACE test®, close the lid, and press the Start/Stop button. When you hear a beep and the colon starts flashing, incubation begins. While incubating, you can hold down the Start/Stop button to pause the timer.

5. Once incubation is finished, the results will appear on the display. A beep will sound: P1 beeps once, P2 beeps twice, ... P9 beeps nine times.


6. Press any key to return to the previous screen. Press the Start/Stop button to restart the countdown timer.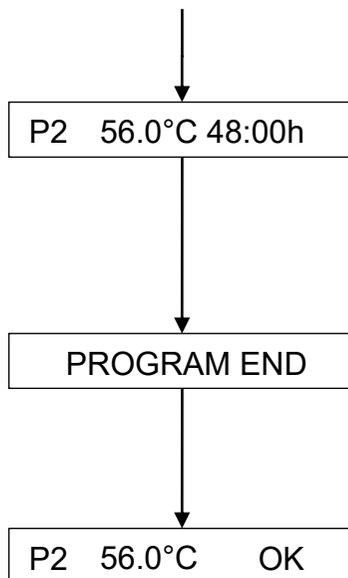

Adjusting Temperature and Time Settings

Press the Prog. button to select the program you want to modify.

Press the ➡ button to choose the digit you want to set, then use the △ or ▽ buttons to adjust the temperature or time. Your changes will be saved automatically after 3 seconds.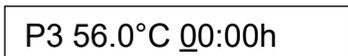

Temperature Calibration

The device is temperature calibrated at the factory. However, if there is any discrepancy between the actual temperature and the one shown on the display, please follow the steps below to adjust it.


For greater accuracy, temperature calibration is performed linearly at two points: 40°C and 80°C. After two-point calibration, the temperature margin of error is within $\pm 0.5^\circ\text{C}$.

When calibrating the temperature, make sure the ambient temperature is below 35°C.

Note: The unit is calibrated at the factory, so please do not use the calibration function during normal operation.

Calibration Procedure

1. Once you turn on the switch, the device enters standby mode. At this point, please make sure the temperature displayed on the screen is below 35°C. If the temperature is 35°C or higher, wait until it drops below 35°C before proceeding to the next step.
2. Pour paraffin oil into the well at the center of the aluminum block, then insert the thermometer into this well (make sure to use a thermometer accurate to within $\pm 0.1^\circ\text{C}$, and that its sensing bulb is fully submerged in the well). At this point, insulate the area from the surrounding temperature using the provided insulation material (see diagram below).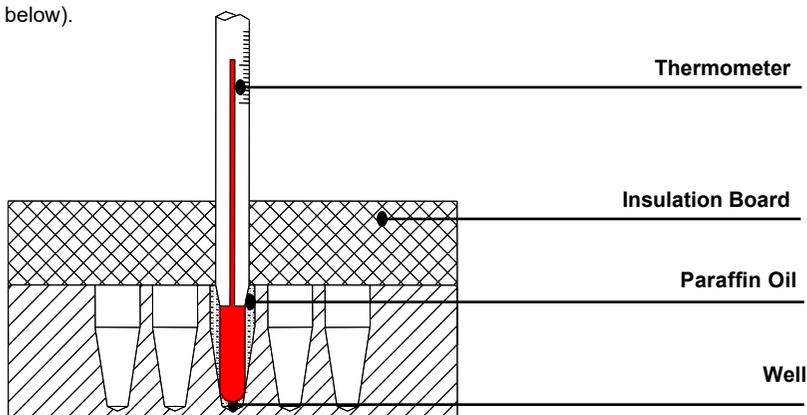


3. Press and hold both the “△” and “▽” buttons at the same time to instantly raise the temperature to 40.0°C. At this point, the “*” indicator will start flashing. When the temperature reaches 40.0°C, both “Adj” and “*” will flash at the same time.

4. After 20 minutes, if the thermometer reads 38.8°C, press the “ ” and “ ” buttons and adjust the value shown to 38.8. Once it displays 38.8, press the “Start/Stop” button to confirm. After confirming, the temperature will automatically rise to 80.0°C and the “*” indicator will flash.

5. When the displayed temperature reaches 80.0°C, both “ADJ” and “*” will flash together.

6. After 20 minutes, if the thermometer shows 79.0°C, press the “ ” and “ ” buttons and set the displayed value to 79.0. Then, press the “Star/Stop” button to confirm.

7. After temperature calibration, the display will show the temperature of the aluminum block.

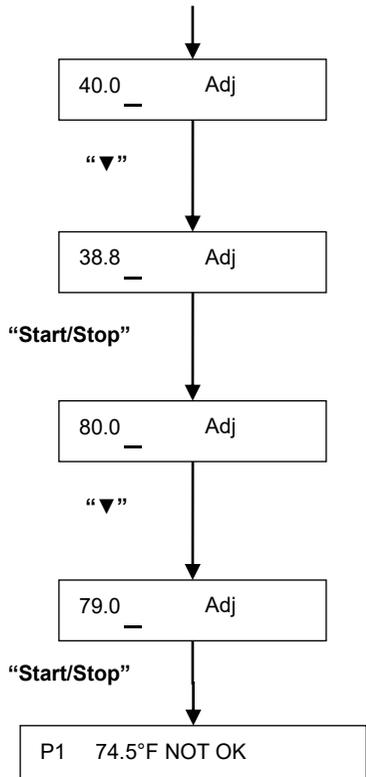Caution:

1) When the temperature reaches 40°C (or 80°C), and both “ADJ” and “*” start flashing, please calibrate after 20 minutes.

2) If you press both “▲” and “▼” buttons at the same time during calibration, the operation will be canceled and the previous calibration data will remain saved.

Troubleshooting Guide

Issue	Possible Cause	Solution
No display on screen	Power cord connection	Please make sure the power cord is securely plugged into the outlet
	Power switch malfunction	
	Controller malfunction	
"OPEN" appears on the display and the buzzer	Sensor malfunction or loose module	Please contact your local distributor.
"SHO" appears on the display and the buzzer sounds	Temperature Controller SHORT	
The block temperature does not rise	Heater malfunction	
The keyboard buttons are not responding	Button malfunction	

Maintenance

To keep things clean, regularly wipe the inside of the block wells with a soft cloth or cotton swab lightly moistened with anhydrous ethanol.

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