

Feces sampling Sheet for gut microbiota analysis Instruction Manual

“Feces sampling Sheet” shown in the lower picture is used to collect stool. The sheet can be treated easy and clean to flush into the toilet after use. Pledase read the precautions and before use.



Precautions

■ Directions for use

- Place “Feces sampling Sheet” in a toilet bowl and defecate.
- Feces should be collected as soon as possible (about 5 minutes).
- Flush with water with the sheet after defecation.

■ Precautions for use

- Float the sheet into the toilet bowl immediately before urinating.
- Use a small amount of toilet paper for smooth flushing.
- Do not use a toilet shower until stool has been collected.
- Do not collect faces during menstrual periods.