


## Instruction sheet

### -For gut microbiota analysis- Fecal Sample Collection Procedure (Brush Type)

#### Feces sampling kit for gut microbiota analysis with preservative solution (brush type / feces sampling sheet)

Cat.No.TSG-FS-0017

Please confirm the contents before use.

|  |         |   |
|--|---------|---|
| Sampling container (Cat.No.TSG-FS-0016-10)                               | 1 piece |  |
| Feces sampling sheet<br>(Feces sampling sheet can be flushed after use.) | 1 sheet |   |
| Instructions sheet   | 1 sheet |   |

Distributor



COSMO BIO USA.

Web : <https://www.cosmobiousa.com/>  
e-mail : [support@cosmobiousa.com](mailto:support@cosmobiousa.com)  
Phone : +1-760-431-4600 Fax : +1-760-431-4604




### Notice

- Use according to the recommended procedure. (Do not use by directly inserting into the anus.)
- If the container has cracks or damage, do not use the product.
- If the preservation solution accidentally gets on your skin, mucous membranes, or wounds, or gets into your eyes or mouth, rinse thoroughly with water. Although it is not immediately harmful, if irritation persists, please consult a doctor as needed.
- As the tip of the brush is thin, please handle with care.
- Store out of reach of infants and pets.
- Do not open the lid except when in use. Close the lid immediately after use.
- Store the container in a cool, dark place (room temperature: 30°C or below), or in a refrigerator, in an upright position.

1


Place "Feces sampling Sheet" in a toilet bowl.

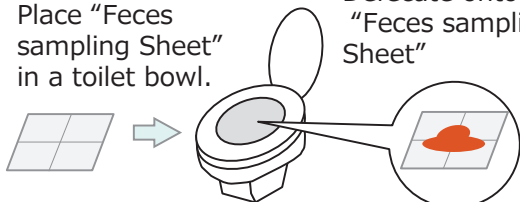

Defecate onto "Feces sampling Sheet"

2


Turn the handle

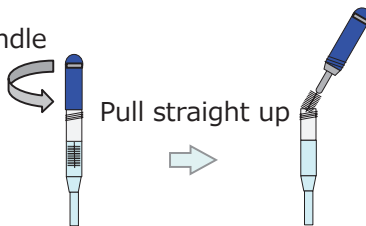

Pull straight up

3


Scoop from multiple areas of the stool.

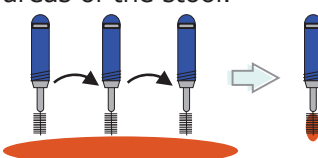

Recommended sample size:  
About a rice grain

4


Return the brush into the tube.

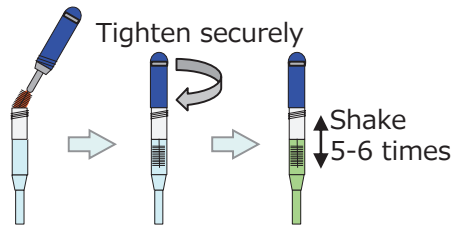

Tighten securely

Shake 5-6 times

5

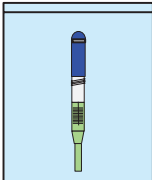

Store the sample in the provided plastic bag, in an upright position, in a cool, dark place or in a refrigerator until submission.

Please read the "Instructions for Use" on the "Feces sampling sheet" carefully before preparation.

#### Please note

Be sure to urinate before you place the stool collection sheet in the toilet bowl.

Hold the sampling container in an upright position, keeping the narrow end down, to prevent spilling the liquid in the tube.

Collect feces on the brush by stabbing the center of the sample several times in different places.

※ Do not take too much feces or the container will overflow.

Ensure proper tube closure.

※ Avoid feces contact with the rim of the container.

※ Tighten the lid securely to prevent any leakage.

Shake the container 5 or 6 times to mix the contents.

Store the sampling container in the provided plastic bag, in an upright position, in a cool, dark place (Room Temperature: below 30°C) or in a refrigerator.